

WayForward Technologies



You should have received the following items:

- **Boat Race™** Installation CD
- **EEGer™** Keyblock File Disk (license update, if games purchased separately)

In order for Boat Race to run, you will be installing game software on the Game computer and a license file on the Therapist computer. Boat Race requires 2-computer or Dual Monitor configuration.

System Requirements for Boat Race

- **EEGer** software version 4.2.0 or greater
- Windows XP/Vista
- 128mb or greater video memory; min. 800x600px, 16-bit color
- DirectX Version 9.0c or later with d3dx9 components*

* **IMPORTANT:** Before installing a newer version of DirectX, be certain your video driver is compatible with that version of DirectX or you may experience problems with displays. DirectX 9 and d3dx9 installations are accessible by Exploring the installation CD contents.

Boat Race Installation



Game Computer – Software Installation

1. If the **EGS** game menu is displayed, press **Esc** to exit to Windows.
2. Insert the installation CD into the Game computer's CD/DVD drive.
3. In just a moment, the CD should start automatically and ask if you would like to install Boat Race. (If the CD does not start automatically, double-click on the "My Computer" icon, and double-click on the CD/DVD drive icon.)
4. Click "Install Boat Race" and follow the on screen installation steps, choosing the default options.
5. Once the software has installed, be sure to restart the computer.
6. If using **EEGer/EGS** Version 4.2.x or greater, finalize the game installation:
2-Computer Configuration: Launch **EGS** and press **F9**. When the list of installed games appears, click **DONE-Save Configuration**.



Therapist Computer - License Keyblock File Installation

1. Insert the **EEGer** Keyblock File Disk into the appropriate disk drive.
2. Double-click the **EEGer** icon to start the program.
3. Select **Files – Read EEGer Keyblock Files**.
4. When prompted, use the Open dialog to locate the keyblock file on the CD. Click the filename to highlight the file and click OK.
5. Finalize the Games:
 - **Dual Monitor Configuration:** select **Tools-Game Initialization Tool**. Click **Done Save Configuration** then **Save All Games**.
 - **2-Computer Configuration:** In **EEGer**, select **Tools – Retrieve Installed Games List**, then **Tools-Tailor Installed Games** and click **SAVE All Games**.

Boat Race Game Description

As the name suggests, Boat Race is a game where three boats race. The positions of the boats are driven by the inhibit and reward band amplitudes, with perspective and wake showing amplitude and speed increasing with amplitude.

- Inhibit conditions cut off the audio and thrust of the center ('reward') boat making it fall back in the race.
- The colored bars at the bottom of the screen correspond to the amplitude of the inhibit and reward bands.
- The fuel tank at the upper-right of the screen indicates how long optimal focus is being sustained (maintaining above threshold reward signal with no inhibits).
- Inhibits can create a fog and dim the sky.
- With sustained focus the reward boat jumps a ramp for added interest.

Game Configuration Options:

1. Click **Start** and select **Programs** or **All Programs**.
2. From the **WayForward** Program Group, select **Boat Race - Configure Boat Race**.
3. Select the appropriate tab to modify settings. Click **Save and Quit** to save changes.

