



EEGer™ Neurofeedback Software

Common Keystrokes

NO BAND SELECTED		
Function	Action	DOS Key
PgUp / PgDn	Select band to adjust	PgUp/PgDn
F1	Help (F1 again to exit)	F1
F2 / Shift+F2	Cycle for Downtraining or other reward mode	Ctrl+F2
F3 / Shift+F3	Switch Channel (A to B), cycle or popup feedback mode menu	F3
F4	Switch protocols (next Stage)	F4
Shift+F4	Turn auto-staging (timer) back on	
F5	Start/Stop	F5
Ctrl+F5	Turn off automatic timer – Manual run	
F6	Save 16 second EEG display to print	F6
Ctrl+F7	Impedance for J&J amplifiers (Ctrl+F7 to return)	
F8	Open/Close EDITOR	
Shift+F8	Insert marker (optional text)	
F9	AutoScale training bands (no band selected)	F2
F10 / Shift+F10	Increase/Decrease spectral display band height	
Ctrl+F10	Hide spectral bands	
F11	AutoThreshold all traces (no band selected)	F11
F12	Graph of live session trend and spectral mirror (F12 to return)	
Alt+F12	Hide unused raw trace wave	
Ctrl+↑ or ↓	Increase/Decrease REWARD frequency	
Alt-A / Alt-B	Insert text note if protocol changed	
Esc	End session and exit	Esc
Space	Toggle single-computer Therapist/Game display	
% or G / V / T / O	Adjust Goal, Volume, Tactile, Options settings	
D / F	Adjust Therapist Display or Feedback Display options	
Alt-A / Alt-B	Change input channel site	

BAND SELECTED (Highlighted)		
Function	Action	DOS Key
PgUp / PgDn	Select band to adjust	PgUp/PgDn
+ / -	Increase/Decrease 1. scale value (press F9, then + or -, Enter) 2. threshold width	+ / -
Ctrl+← or →	Increase/Decrease LOWER frequency for band	
Ctrl+↑ or ↓	Increase/Decrease UPPER frequency for band	
F9	Manually set scale for currently selected band	F2
F11	AutoThreshold for selected band	
Enter	Finish change on current band	Enter



EEGer™ Neurofeedback Software Expanded List of Keystrokes

NO BAND SELECTED		
Function	Action	DOS Key
PgUp / PgDn	Select band to adjust	PgUp/PgDn
F1	Help (F1 again to exit)	F1
F2 / Shift+F2	Cycle for Downtraining or other reward mode	Ctrl+F2
F3 / Shift+F3	Switch Channel (A to B); cycle or popup feedback mode menu	F3
Ctrl+F3	Toggle game color	automatic
Alt+F3	Toggle Low Pass filter	
F4	Switch protocols (next Stage)	F4
Shift+F4	Turn Autostage back on	
Ctrl+F4	Change to previous session frequency	
F5	Start/Stop	F5
Ctrl+F5	Turn off automatic timer – Manual run (Shift+F4 to turn back on)	
F6	Save current EEG display to print	F6
Ctrl-F6	Change feedback source	
Alt+F6	Sine wave signal generator	
F7	Toggle integral display (debug)	
Ctrl+F7	Impedance for J&J amplifiers (Ctrl+F7 to return)	
F8	Insert comment (text)	
Alt+F8	Toggle Arti Inhib	
F9	AutoScale training bands (no band selected)	F2
Ctrl+F9	Single scale A-adjust	
F10 / Shift+F10	Increase/Decrease spectral display band height	
Ctrl+F10	Hide spectral bands	
Alt+F10	FFT markers On/Off	
F11	AutoThreshold all traces (no band selected)	F11
Ctrl+F11	Toggle AutoThreshold at stage and AutoThreshold with F11	
F12 / Shift+F12	Graph of live session trend and spectral mirror (F12 to return)	
Ctrl+F12	Toggle draw speed	
Alt+F12	Toggle “hide unused traces”	
Ctrl+↑ or ↓	Increase/Decrease REWARD frequency	
Alt-A / Alt-B	Insert text note if protocol changed	
Esc	End session and exit	Esc
Space	Toggle single-computer Therapist/Game display	
% or G / V / T / O	Adjust Goal, Volume, or Tactile settings	
D / F	Adjust Therapist Display or Feedback Display options	

BAND SELECTED (Highlighted)		
Function	Action	DOS Key
PgUp / PgDn	Select band to adjust	PgUp/PgDn
+ / -	Increase/Decrease: 1. scale value (F9, then + or -, Enter) 2. threshold width	+ / -
Ctrl+← or →	Increase/Decrease LOWER frequency for band	
Ctrl+↑ or ↓	Increase/Decrease UPPER frequency for band	
F9	Manually set scale for currently selected band	F2
Ctrl+F9	AutoScale currently selected band	
F11	AutoThreshold for selected band	
Enter	Finish change on current band	Enter